# INSTRUCTIONS FOR INSERTION AND USE

The intrauterine contraceptive device GINETIK is indicated for contraception, used to prevent from pregnancy. It is made of polyethylene of the adequate quality. Contrast agents are also added, so that x-rays can monitor its position. A body of the device is coiled with copper wire in a total surface of 250 mm<sup>2</sup>, insuring a very reliable contraceptive protection. The probability of conception is less than 1%. The efficiency of the device lasts up to 5 years.

## INSERTION OF THE GINETIK:

Gynaecological examination should be performed before the insertion of this contraceptive device to determine suitability of the uterus for this type of contraception (position and shape of the uterus, possible anomalies and diseases).

The physician must introduce both, risks and benefits associated with the use of this contraceptive product to the patient prior to insertion.

The device can be inserted anytime during the menses cycle, the most convenient, however, are the last days of the menses. The device can be also inserted after an abortion.

In order to insert this contraceptive device properly, it should be pulled into the insertion tube with a string only so far as to enable the sides of the contraceptive device to be fully enclosed into the insertion tube, but the ends of each side should be left outside of the insertion tube and should form a rounded top to enable easier insertion through the cervical canal. Rounded top makes the insertion easier. Since the ends of each side of the contraceptive device may hide into the insertion tube, the device should therefore be retained with the flange during the insertion. The device can get damaged if it is ousted from the insertion tube before time. Device should not be released from the insertion tube with solid rod until it reaches the appropriate depth in the uterus. Insertion tube should be moved slowly from uterus simultaneously with the elimination of the device from the insertion tube. Otherwise the device could get damaged. This procedure will enable the placement of the device at the highest possible position within the endometrial cavity. The flange should be adjusted in order to indicate the depth to which the device should be inserted and the direction in which both sides of the device will open.

After the insertion of the device, the strings must be cut so far that they stretch into the vagina for about 2 cm. The strings facilitate the checking of the presence and position of the device and make its removal easier. The device can be removed by the slow traction of the strings.

**GINETIK** is sterilized with etilenoxid and intended for a single use only. It is made in two sizes:

- **GINETIK normal** is designed for women, who have given birth before
- **GINETIK mini** for women, who have not given birth yet

The device is convenient for women who are not planning pregnancy anymore.

## CONTRAINDICATIONS:

The only absolute contraindication is an acute or subacute inflammation of the true pelvis and possible existing pregnancy. Among the relative contraindications can be ranged congenital or acquired uterine malformations, larger myoms, menopause, metrorragia and anaemia.

## COMPLICATIONS:

After its insertion, the device may cause more or less expressed pain in the lower part of the abdomen and lumbago, usually not of long duration. The following menses can be somewhat stronger and longer. Intermediate bleedings can occur, usually stopping quickly. There's a possibility of extrauterus pregnancy. Certain inflammations may also occur that needs to be cured adequately.

## STORAGE AND HANDLING THE PRODUCT:

The items should be stored in dry, clean and suitable aired store. One should be careful that the basic package doesn't get damaged, otherwise the sterility is not assured.

## WARNING:

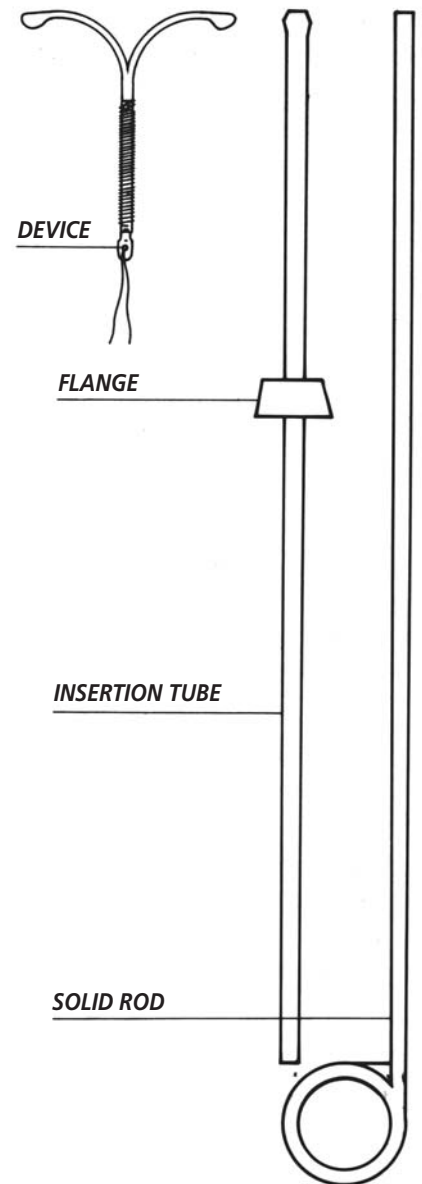
- Only a specialized physician should perform the insertion of the device. After the insertion of the contraceptive device a gynaecological checking after three, six and twelve months is recommended. And thereafter a gynaecological checking should be performed once a year, or if certain irregularities or troubles occur. It's recommended for the women to check the presence of the device after each menses by themselves.
- The batch code is used for traceability. The code needs to be registered after the insertion of a device.
- Re-sterilization is not allowed.
- Ginetik is not appropriate for women allergic to copper.

The contraceptive device GINETIK has been tested by the Gynaecological Clinics of Ljubljana. The permission to use CE sign was acquired in 2004.

## INTRAUTERINE CONTRACEPTIVE DEVICE

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